

Cutting of a 2cc Syringe for Use as a Carrier of Irradiated Bone



Cutting the End



The Cut Syringe



Use Syringes that are not too Hard



Hard Syringes are too Hard
to cut



The Plastipak Syringe



The Plastipak Syringe



Draw Back the Plunger



Draw Back the Plunger



Point the Blade away for Safety



Point the Tip of the Blade at Right
Angles to the Syringe



Point the Tip of the Blade at Right
Angles to the Syringe



Ease the Tip of the Blade into the
Syringe



Move the Tip of the Blade Backwards
and Forwards




Keep pushing the Tip into the Syringe







The Tip of the Blade through the Side
into the Syringe

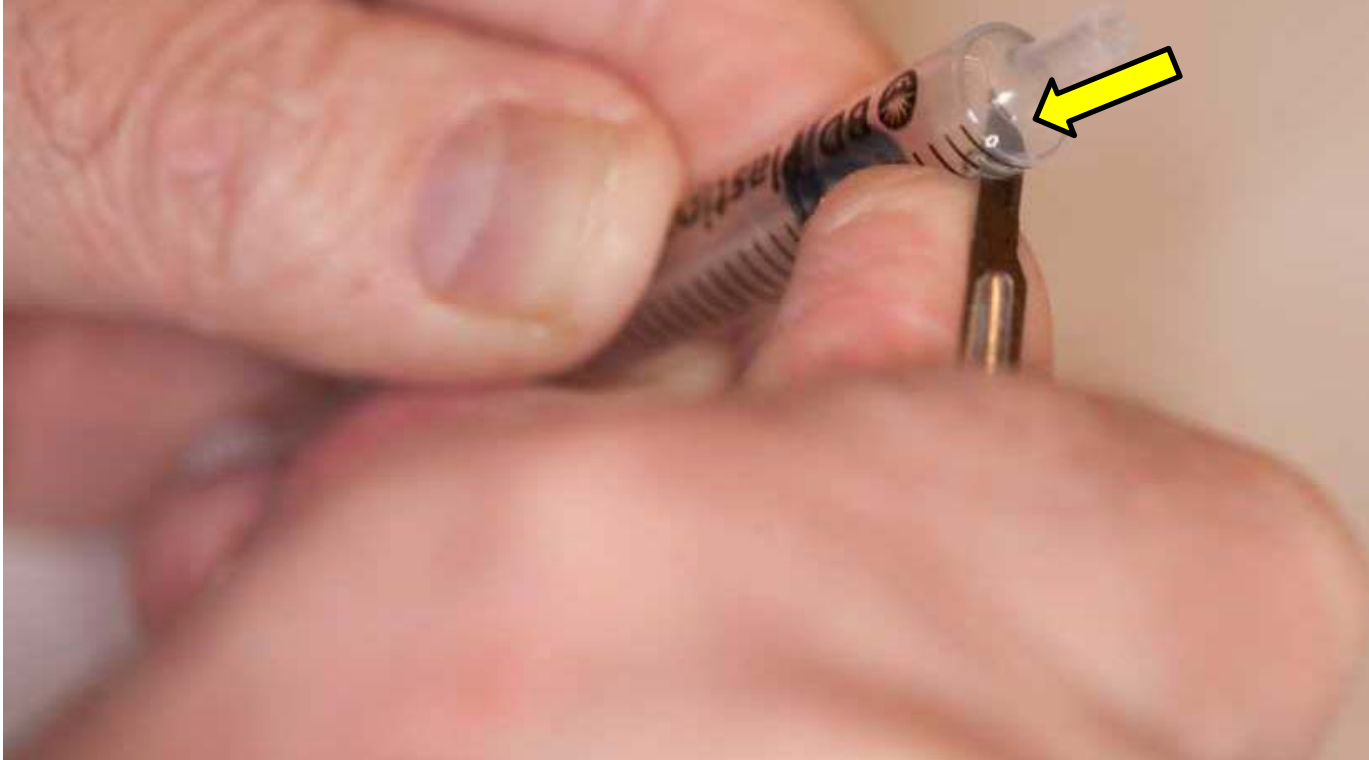

The Tip of the Blade through the Side
into the Syringe

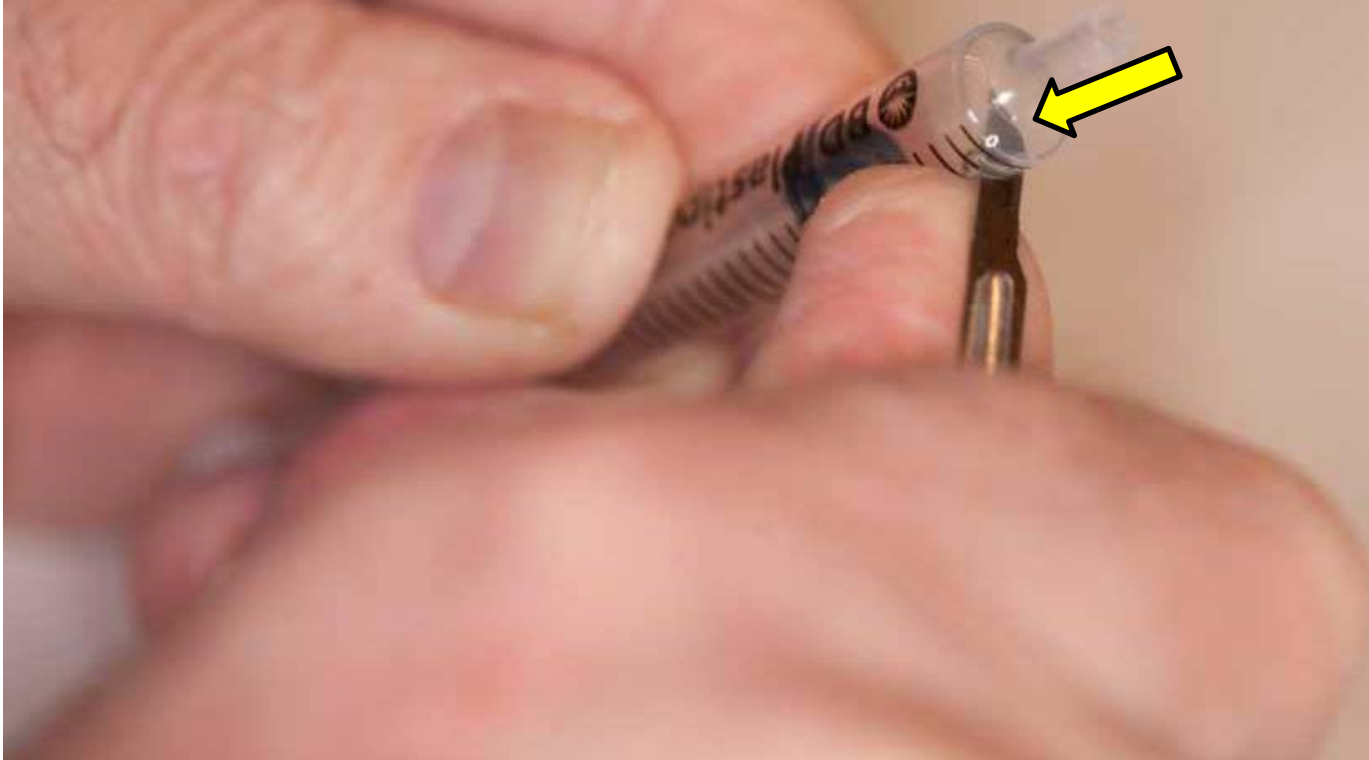

Turn the Syringe Clockwise

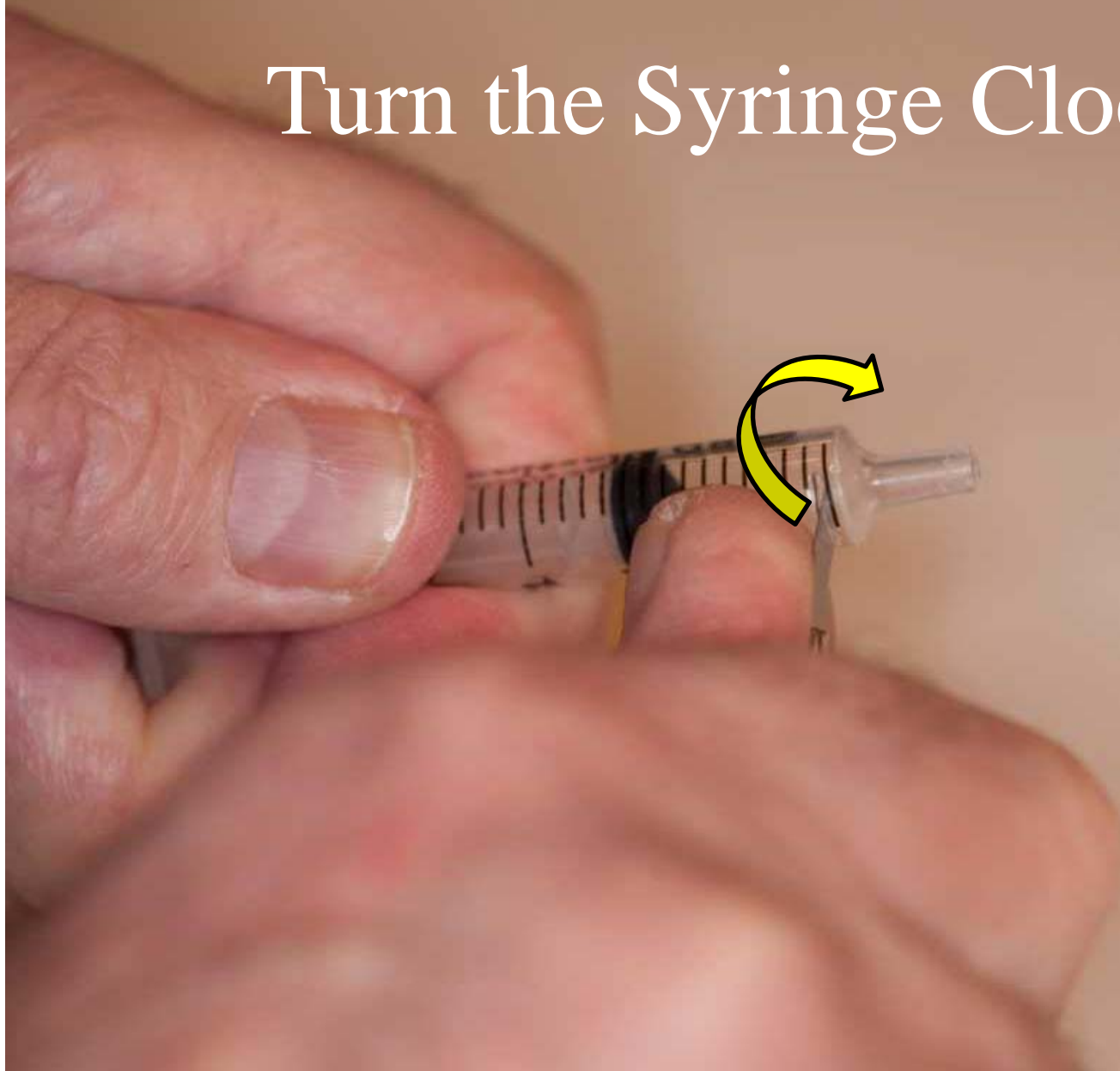

Push the Blade away



Cut round the Syringe










Pushing away is Safe



If the Blade slips out
it will not cut the Operator



Re-insert the Blade

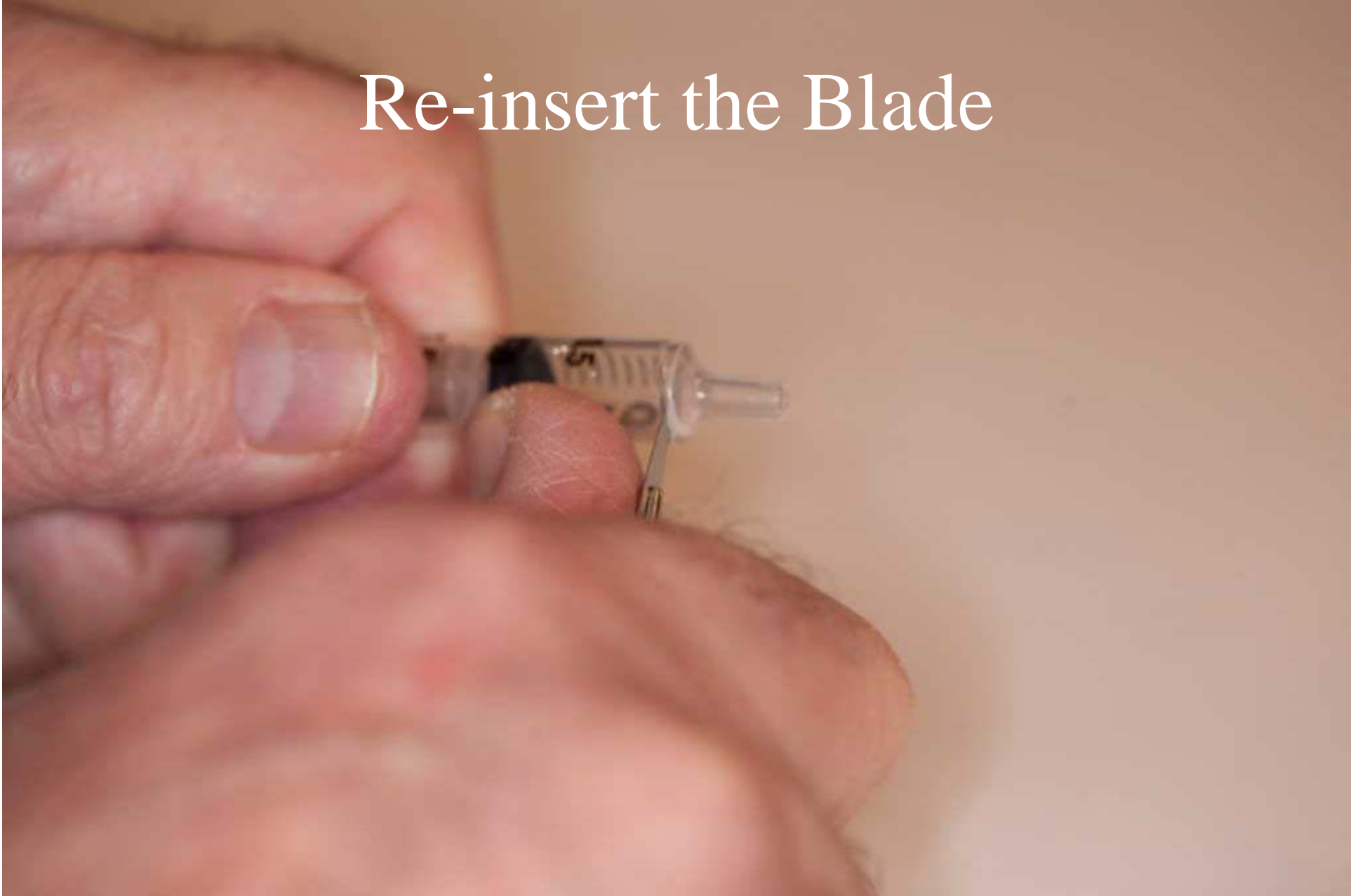



Continue cutting round the Syringe



The Tip of the Syringe being cut off



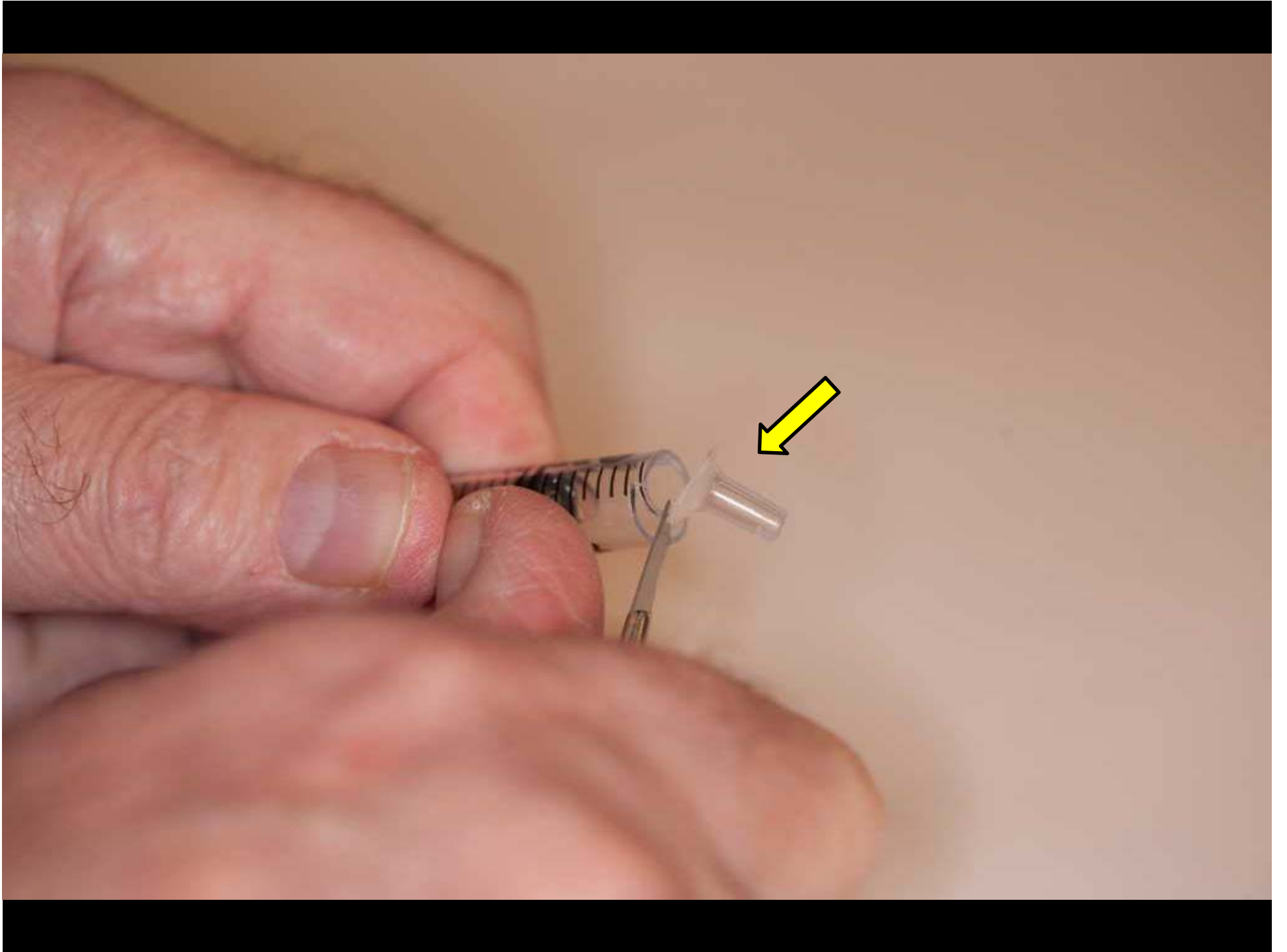

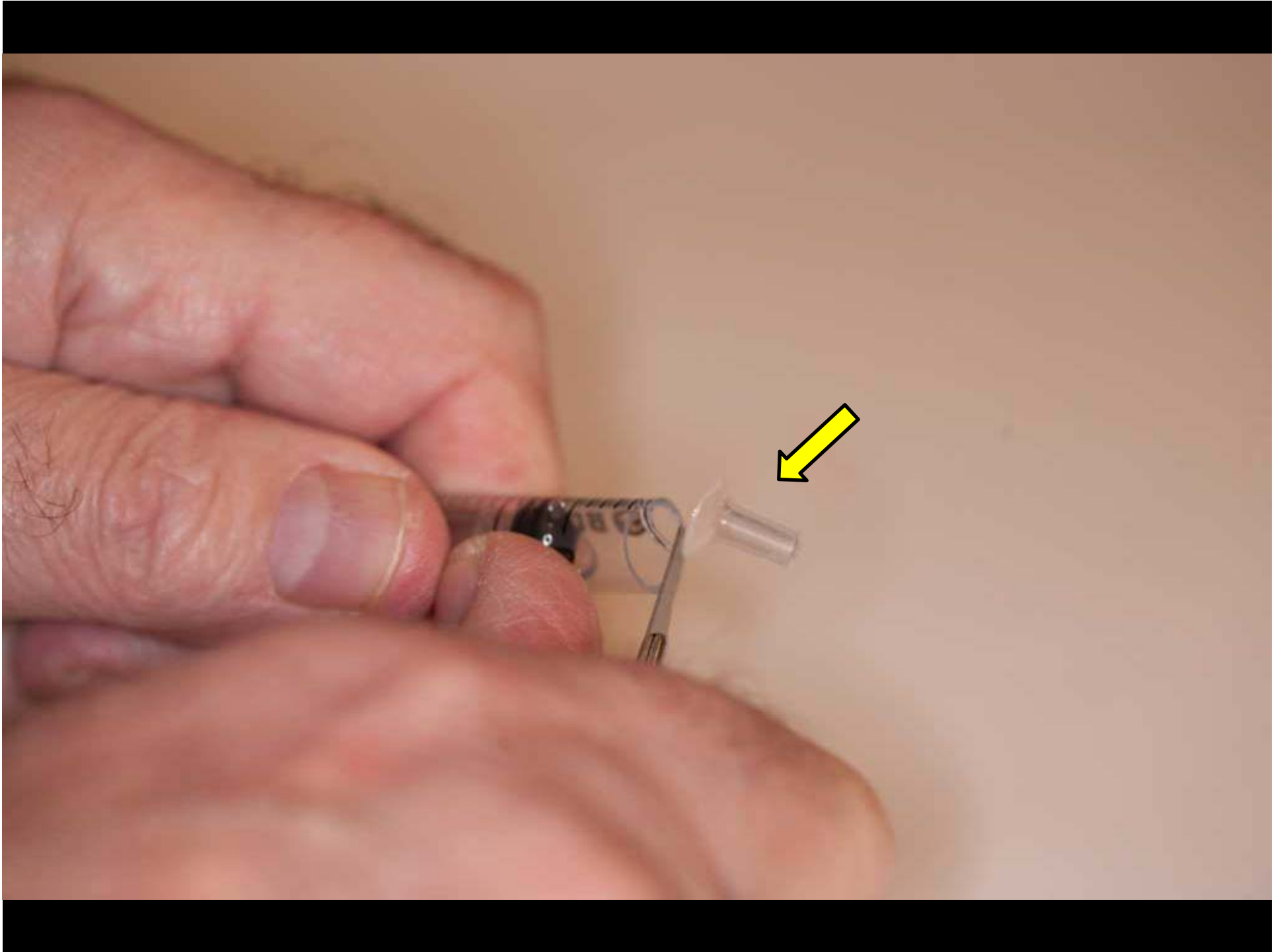

The Cut off Syringe



During Use the Rubber Tip may be pushed through the End by Mistake



Remove the Rubber Tip



Draw Back the Plunger





Place the Rubber Tip



The Rubber Tip replaced



Replace the Plunger



Replace the Plunger



The Syringe Ready to use



End